

Topi-Pump

“Metered Dosing for Topical Creams and Gels”

Assembly Information Sheet

Specifications:

- The **Topi-Pump** has four parts - Housing, Pouch, Actuator and Overcap.
- The housing and overcap are light resistant amber in color.
- The collapsible pouch comes in three sizes **35mls**, **65mls** or **130mls** when filled.
- The actuator dispenses **1ml (or .5mls)** with each pump (+/- 2%).

Assembly:

1. Fill pouch with formula to be dispensed (Do not exceed bottom of the neck).



2. Next place the actuator into the opening of the pouch and using the heel of your palm, press down firmly on the actuator until it snaps into the pouch.



3. Now prime the pump by pressing down on the actuator until some contents are dispensed. Once a small amount of the contents are expelled - the next pump will be a full dose.



Warnings: Topi-Pump should be evaluated by a Pharmacist to insure suitability for prescription medications.

Tips:

- When filling the pouch be sure to clean off any excess material spilled on the “o” ring seat as this can cause a failure to prime.
- The pouch will dispense 99% of the contents however, on the last few pumps the actuator may take a few minutes to draw up the full dose. This can be observed by watching the actuator slowly return to the full upright position.
- Getting the actuator assembly to properly seat into the housing is the critical step. You will notice when you press down on the actuator there is a distinct click.